



**MARS**<sup>TM</sup>  
BIO-MED PROCESSES INC

## The "Best of Both Worlds" Combo Package

**POWERLINZ**<sup>®</sup>  
CONCENTRATE

**CLEANLINZ**  
CONCENTRATE



Daily Dental Evacuation Line Cleaner



Quarterly Dental Evacuation Line Cleaner

### Benefits:

- Anti-bacterial solution kills bacteria and reduces bacteria load in the lines & in the amalgam separator
- Pleasant mint scent helps to remove unpleasant odors in operatories and offices
- Highly concentrated solution reduces the mercury load in the evacuation system
- Non-foaming and neutral pH, safe to use with all pump systems
- Chlorine-free and non-corrosive
- Convenient pre-measured pump for ease of use

### Benefits:

- Has a bacterial component (which daily line cleaners do not have) that "eats away" at tough, stuck-on sludge which builds up in the evacuation lines over time
- Removing this sludge opens up the diameter of the lines, allowing the vacuum pump to work more efficiently while improving suction
- Non-foaming and safe to use with all pump systems
- Highly concentrated solution reduces the mercury load in the evacuation system
- Convenient pre-measured chamber for ease of use

### When To Use:

- PowerLINZ (the daily cleaner) is used once daily, at the end of the day, when CleanLINZ is not in use.

- CleanLINZ (the quarterly cleaner) is used once daily, at the end of day, for an 8-day cycle. This 8-day cycle is repeated every 3 months.

\*Please Note: PowerLINZ should be used daily, except during the 8-day CleanLINZ cycle.\*

### Having suction issues?

The combination of these two super concentrated formulas eliminates sludge & reduces odor, while improving the system's suction capacity.