


Quarterly Promotions!



Intra-oral Adjustment/Polishing
for Zirconia and Lithium Disilicate

Adjust and Polish Zirconia and
Lithium Disilicate the **eZr™** Way!

eZr™ Intra-oral **Adjustment:**

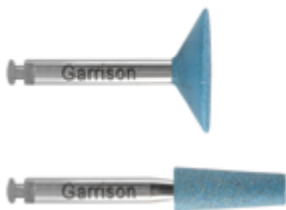


- **No chatter, chipping or microfracture**
- Unlike diamonds bonded directly to a hard metal shaft, eZr's rubber substrate cushions the diamonds to prevent damage to the restoration.
- **No heat build-up** - The carefully engineered substrate does not transfer heat during use.
- **FAST!** - eZr's coarse grit blue grinders make your adjustments fast and convenient.

Kit	Qty.	Price
FPZK01 Complete system kit	2 grinders 6 polishers	\$280 ⁹⁹

Canadian MSRP



1 Start by gross adjusting and contouring using the blue coarse grit grinders. Use the flat end taper for occlusal/labial/lingual/buccal surfaces and the disc for interproximal surfaces.

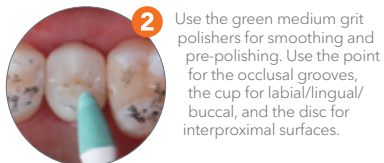

2 Use the green medium grit polishers for smoothing and pre-polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.



3 Use the orange fine grit polishers for high gloss polishing. Use the point for the occlusal grooves, the cup for labial/lingual/buccal, and the disc for interproximal surfaces.



Beautiful
results!

eZr™ Intra-oral **Polishing:**



- **Unparalleled smooth polish**
- Don't take our word for it! - "The eZr system is the only one I've tried that polishes zirconia to smooth!" Todd Brower, DDS, Holland, Michigan.
- **Increased physical stability** - Highly polished zirconia and lithium disilicate is stable and non-abrasive to opposing dentition.
- **No paste required** - Forget the mess and expense of polishing pastes. eZr™ is all you need.
- **No water required** - No need to use water when polishing. Water is only required for high speed.



Slow speed right angle latch shank
Steam autoclaveable

Buy a **eZr kit (FPZK01)**
Get a **fine grit small point polisher (FPZF010) 5 pack**
> FREE! <

\$77 CAD value

*To redeem goods, email a scanned copy of your dealers invoice dated 1/10/25 - 31/12/25 to: Garrison Dental Solutions, Attn: Liz Kammers to ekammers@garrisdental.com with INCA25Q4 in the Subject line. Allow 4-6 weeks for delivery.

Garrison



www.dfdentalsolutions.ca | 1 (866) 740-8829

INCA25Q4

© 2025 Garrison Dental Solutions, LLC